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Regolamento UE2016/425

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Regulation requirements

Regulation UE2016/425

requirements for designing and manufacturing PPE;

UNI EN 166:2004 - Personal protection of the eyes

Specifications, CE 765/2008 - Graphic signs - Colour and safety signs

Recorded safety signs.

Istruzioni d'uso

ATTENTION/WARNING: put the PPE on before exposing yourself to risks

Markings:

Markings on the lenses: L 1 3 S CE - 1 (OPTICAL CLASS), 3 (FIELD OF USE),
S (INCREASED ROBUSTNESS)

Markings on the frame: L EN 166 3 S CE - S (INCREASED ROBUSTNESS),
EN 166 (PRESENT LEGISLATION)

Larident Srl

Via Lamaneigra 12R (Ponte Rimassa)

16030 Tribogna (Genova)

Tel. +39 0185 939481 - Fax. +39 0185 938595

info@larident.it - www.larident.it

PROTECTIVE VISOR R52 INSTRUCTIONS

The protective visor is classified as PPE class II, it protects the face from drops/sprays of water and saliva and reduces risks of infection.

Very comfortable with characteristics that mean it can be worn safely and without any annoyance for hours.

It can be worn with reading glasses due to the removable nose clip.

Completely transparent polycarbonate protective shield that can be replaced easily and quickly.

Larident Srl

Via Lamaneigra 12R (Ponte Rimassa)


16030 Tribogna (Genova)

Tel. +39 0185 939481 - Fax. +39 0185 938595

info@larident.it - www.larident.it

PROTECTIVE VISOR R52

INSTRUCTIONS FOR ASSEMBLY

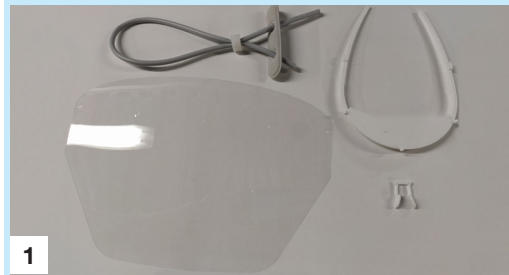
1
The components of R52


2
Remove the protective film from both sides


3
Tilt the frame so that the slot is on the lower part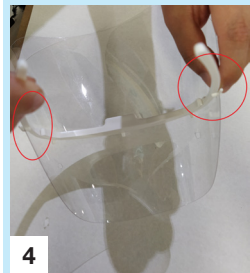


4
Insert the two most external visor holes into the plastic pins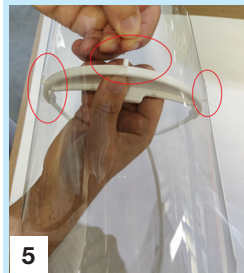


5
Gradually pull the visor up wards clicking the last 3 pins into the holes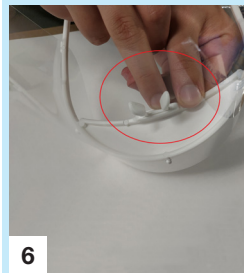

6
The kit contains a nasal support that can be used when necessary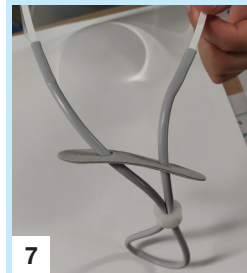

7
The kit contains a silicone elastic strap to be adjusted to the size of the head/non slip

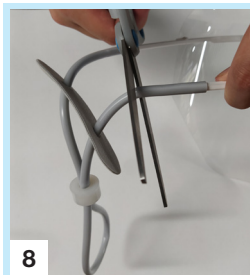8
If you do not wish to use the head strap, cut about 3 cm of it and put it on the end of the frames' legs so the visor is more comfortable

9
The visor is ready

PROTECTIVE VISOR MAINTENANCE R52

- Clean the shield regularly, after every use
- Verify after every use that the shield hasn't got evident damage like scratches, cuts and markings
- Do not carry out cleaning in direct sunlight
- Use absorbent cloths and soft material to dry the shield
- Use tepid water or neutral soap, or any detergent for optical use as a cleaning liquid
- Do not use strong alkaline detergents
- Do not use alcohol

PROTECTIVE VISOR R52

DESCRIPTION

Warnings

The materials used to make our visors can cause allergies to very sensitive skins. Pay attention and handle with care.

Maintenance

Do not use alcohol solutions, they can cause scratches and misting up.

CLEANING

For the shield and the frame use disinfectant, no alcohol solutions based on soaps or other suitable detergents. After cleaning rinse away all residue detergent under running water.

HOW TO USE

- Use for protecting the face from drops/sprays of water and saliva
- Must be used by one person
- It can be used with other PPE, like FFP masks or reading glasses

INSTRUCTIONS: HOW TO WEAR

Put the PPE on and position it correctly on the head. Adjust the strap at the back of the visor to the user's head size. Check, after tightening the strap suitable for the user's head, that the shield fully covers the face including the side of the face near the ears, the forehead and the rest of the face to under the chin. For who uses reading glasses remove the nose clip.

ADVICE

- Accurately examine the PPE before every use, and do not use it if there are obvious signs of it having been previously used or damaged.
- Do not tamper with the PPE.
- Follow the instructions for assembly indicated in the previous paragraph.
- If the PPE shield is damaged or scratched it must be replaced.