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### Normative requirements

Second category personal equipment that conforms to the Regulation UE 2016/425 requirements for the design and manufacture of the PPE;

UNI EN 166:2004 - Personal eye protection

Specifications, CE 765/2008 - Graphic signs - Colors and safety signs  
Registered safety signs.

PPE manufactured according to the laws:  
UNI EN 166:2004 UNI EN 167:2001 UNI EN 168:2001

Instructions for Use

#### NOTIFIED BODY

The protective visors and protective glasses, are category II PPE for this reason they need the issued UE that conforms with the Regulation (UE) 2016/425 from a notified body.

Larident SRL collaborated with the notified body MTIC INTERCERT  
Via G. Leopardi, 14 - 20123 Milan (MI) - Italy

**WARNING:** wear PPE before exposing yourself to risk

#### Marking:

Shield marking: L 1 S CE

L (Larident identification) - 1 (optical class) - S (increased robustness) - CE (marking)

Frame marking: L EN 166 3 S CE

L (Larident identification) - EN 166 (number of standard applied)  
3 (protection against liquid sprays) - S (increased robustness) - CE (marking)

Symbol read the instructions  
carefully

CE conformity marking

The foreseen duration of the  
product is 5 years, while  
un-opened and packaging  
intact, if kept away from  
direct heat.

**Larident Srl**

Via Lamaneigra 12R (Ponte Rimassa)  
16030 Tribogna (Genova)  
Tel. +39 0185 939481 - Fax. +39 0185 938595  
info@larident.it - www.larident.it

**Larident Srl**

Via Lamaneigra 12R (Ponte Rimassa)  
16030 Tribogna (Genova)  
Tel. +39 0185 939481 - Fax. +39 0185 938595  
info@larident.it - www.larident.it

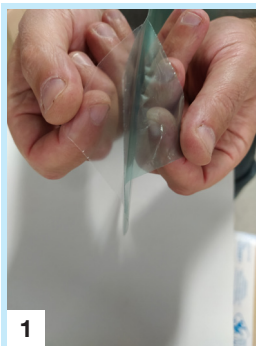
**EN**

## PROTECTIVE GLASSES R58 INSTRUCTIONS

The protective glasses are classified as category II PPE and allow you to protect the face from splashes, waetr droplets and saliva, and reduce the risk of infection. Very comfortable with features that make them wearable for hours safely and without discomfort.  
Protective polycarbonate glasses with absolute trasparency, replaceable quickly and easily.


# PROTECTIVE GLASSES R58 ASSEMBLING

**1**  
Remove the protective film from both sides

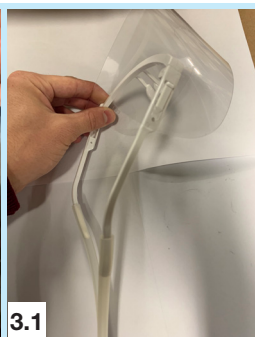**2**  
The glasses must be positioned outside the frame, with the nasal opening downwards

**3**  
Insert the visor into the two side tabs of the frame, fitting it near the hole

**4**  
Remember to adjust the glasses according to the shape of the face so that they fit snugly against the cheekbones

**5**  
The glasses are ready

**ATTENTION: IF THE VISOR IS SCRATCHED OR FOGGED, IT MUST BE REPLACED WITH A NEW ONE.**

# PROTECTIVE GLASSES R58 DESCRIPTION

## Warnings

The materials used to compose our glasses can cause allergies to very sensitive skin. Be careful and handle with care.

## Maintenance

Do not use alcohol solutions, they can cause scratches or fogging.

## CLEANING

For the protective glasses use non-alcoholic disinfectant lotions based on soaps or suitable detergents, the same for the frame. After washing, remove all remnants of the detergent under running water.

## METHOD OF USE

- Use as a face protection from drops of saliva or water.
- Use by single person.
- Can be worn in combination with other non-connected PPE, such as FFP masks or eyeglasses.

## HOW TO WEAR: INSTRUCTIONS FOR USE

Wear the PPE as if they were eyeglasses, checking that the screen covers the entire face of the user and that the upper part of the visor is adherent to the forehead.

For a correct positioning of the visor, check that the nose pad is snug, adjusting the lace according to the size of the user's head.

Once you have verified that the field of view is good and does not blur or impair your vision, the PPE will be worn correctly.

## RECOMMENDATIONS

- Carefully inspect the PPE before each use, and do not use it if any part shows signs of evident wear or malfunction.
- Do not tamper with the PPE.
- Follow the assembly instructions indicated in the previous paragraph.
- If the PPE has suffered damage or scratches to the screen, it must always be replaced.

### Normative requirements

Second category personal equipment that conforms to the Regulation UE 2016/425 requirements for the design and manufacture of the PPE;

UNI EN 166:2004 - Personal eye protection

Specifications, CE 765/2008 - Graphic signs - Colors and safety signs  
Registered safety signs.

PPE manufactured according to the laws:  
UNI EN 166:2004 UNI EN 167:2001 UNI EN 168:2001

Instructions for Use

#### NOTIFIED BODY

The protective visors and protective glasses, are category II PPE for this reason they need the issued UE that conforms with the Regulation (UE) 2016/425 from a notified body.

Larident SRL collaborated with the notified body MTIC INTERCERT  
Via G. Leopardi, 14 - 20123 Milan (MI) - Italy

**WARNING:** wear PPE before exposing yourself to risk

#### Marking:

Shield marking: L 1 S CE

L (Larident identification) - 1 (optical class) - S (increased robustness) - CE (marking)

Frame marking: L EN 166 3 S CE

L (Larident identification) - EN 166 (number of standard applied)  
3 (protection against liquid sprays) - S (increased robustness) - CE (marking)

Symbol read the instructions  
carefully

CE conformity marking

The foreseen duration of the  
product is 5 years, while  
un-opened and packaging  
intact, if kept away from  
direct heat.

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